



# User Manual – FeatherTouch Virtually Painfree Lancing Device

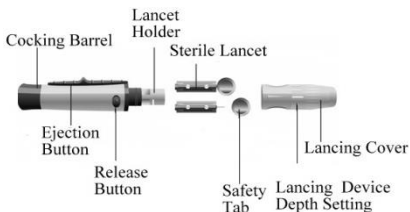

## How to use Feather Touch:

1. Before testing wash your hands with soap water.
2. Unscrew the lancing device cover from the body of the lancing device
3. Insert a fresh lancet firmly into the white lancet holder cup.

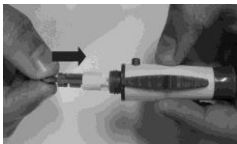

4. Hold the lancet firmly in the lancet holder and twist the




safety tab of the lancet until it loosens. Pull the safety tab off the lancet. Save the safety tab for lancet disposal.

5. Carefully screw the cover back onto the lancing



device. Avoid contact with the exposed needle. Make sure the cover is fully seated on the lancing device

6. Adjust the puncture depth by rotating the lancing

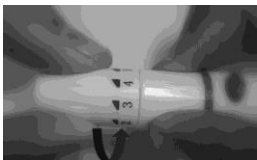

device cover. Feather Touch provides five different puncture depth levels. To reduce discomfort, use the lowest penetration level adequate for

drawing a drop of blood. Higher level indicates higher penetration.

- Use settings 1 and 2 for delicate skin, 3 and 4 for normal skin, and 5 and 6 for calloused or thick skin.
- To lance your finger, we recommend that you start at Level

7. Pull the cocking handle back until it clicks. If it



does not click, it may have been cocked when you inserted the lancet. The device is now ready for pricking the body to derive a drop of blood.

8. Press the mouth of FeatherTouch against the body part and press the



release button to make prick the surface.


## Disposal of a Lancet

1. Unscrew the lancing device cover. Place the safety tab of the lancet



on a hard surface and carefully pierce the lancet needle into the safety tab so that the needle is inserted in the safety tab and is harmlessly unexposed.

2. Press the release button to make sure that the

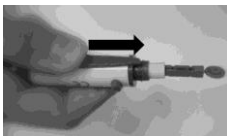

lancet is in the extended position. Slide the ejection button forward and eject the used lancet into a disposal bin. Place the lancing device cover back on the lancing device.

## WARNINGS

- a. Disinfect the lancing device before & after useage
- b. Never reuse a lancet