


TrueHb Test strips

The TrueHb® Hemometer Strips are thin plastic strips which contain chemical reagents that perform our advanced nano-biochemistry. The total hemoglobin level in a blood sample is found using the TrueHb Hemometer Strips on the TrueHb® Hemometer.

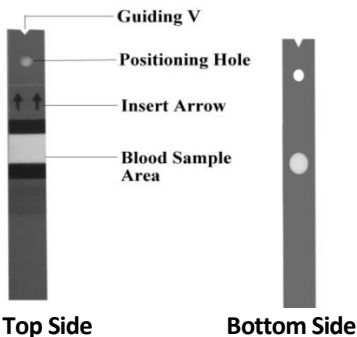

Guiding V- V mark in the strip should be inserted in the test strip holder. The guiding V ensures right positioning of the test strip.


Positioning hole- While inserting the test strip in the device, always make sure that the positioning hole locks with the positioning lock in the device.

Insert Arrow – It shows the direction in which the test strip should be inserted

Blood Sample Area- Apply blood sample

Blood sample Application

Fill the blood sample area with the blood drop (10 μ l). Incorrect result may occur if the blood sample is not applied correctly, or the blood sample area is not covered completely

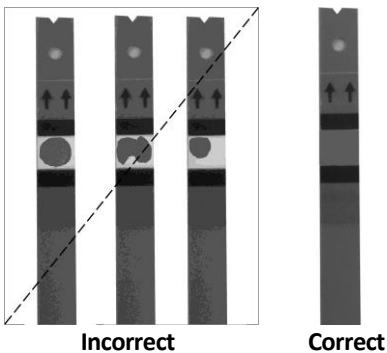

After applying the blood sample ensure the blood sample application area is completely covered. If the blood application area is not covered completely repeat the test with a fresh test strip

Important:

- Use only sample area to apply blood.
- Test strips may be used only once.
- Discard used test strips as per local regulations & blood sample disposal guidelines.

Important TrueHb® Hemometer Test Strip Information

- The TrueHb® Hemoglobin Monitoring System has an operating range of 5°C to 45°C.
- Store the test strip package in a cool, dry place between 0°C to 30°C.
- Do not freeze or refrigerate.
- Use test strips only within the system operating temperature range
- Keep away from direct sunlight and heat
- Store your test strips in their original vial only; never transfer them to another vial or any other container
- Never store individual test strips outside the vial
- After removing a TrueHb® Test Strip from the vial, immediately close the vial cap tightly

- With clean, dry hands, you may gently touch the test strip on blue parts while handling them.
- Do not use test strips beyond the expiration date as this may cause inaccurate results
- Do not bend, cut, deform or alter TrueHb® Test Strips
- Dispose of test strips following local biohazard regulations
- Use test strips only with the genuine TrueHb hemoglobin meter
- For in vitro diagnostic use only.